



- You can clip the tube tightly without making holes on your clothes.

- The clips have lengths of 15mm and 22mm.


- It provides the patient safety and comfort with the clip to turn by following complex physical movement of the patient.

ClipAid™ for Corrugated tube


Model	Tube size	Package unit
ClipAid™ 15	Φ15mm	3 piece/Box
ClipAid™ 22	Φ15mm	3 piece/Box
ClipAid™ button	-	3 piece/bag

ClipAid™ for Corrugated tube


How to use the product

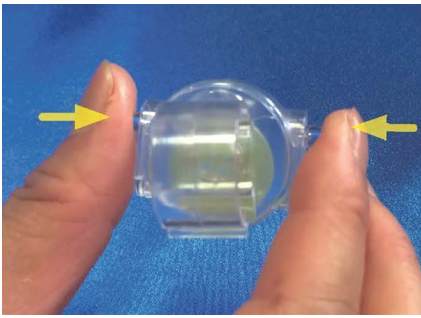


① Press the right and left protrusions to open the holder.

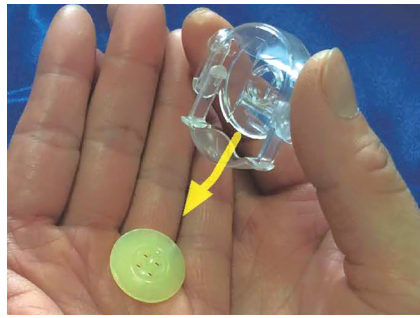


② Pull out the button.

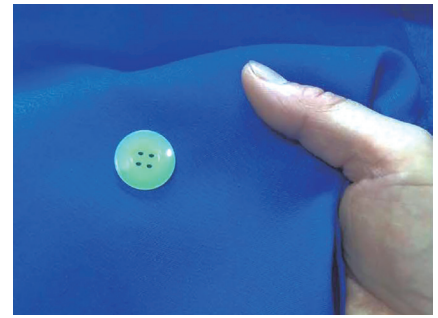

③ Place a button on the back of the fabric.

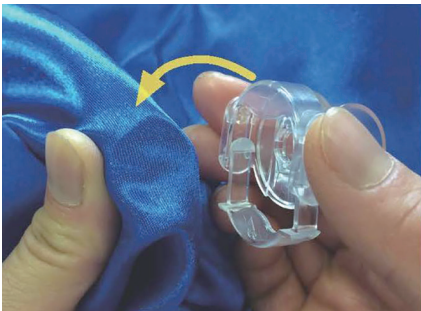


④ Place the holder over the fabric.

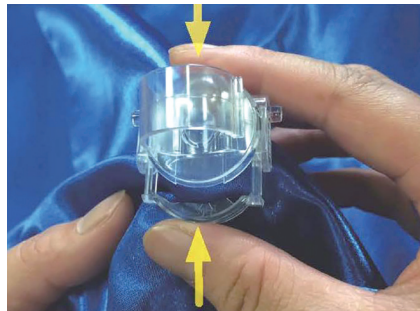

⑤ Press the holder slider firmly.



⑥ Push in until you lock it in.

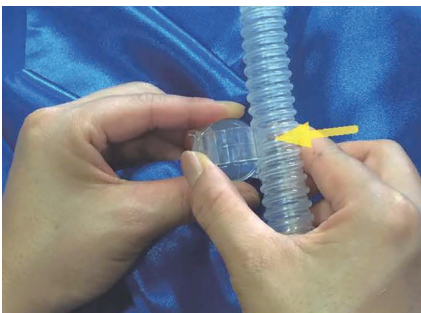

⑦ Insert the flexible tube into the clip.



⑧ Fit in completely.



⑨ Done.

Caution

If Infants/babies use this product, please use it with sewing fixed buttons to prevent accidental aspiration.



Sew a button on the back of the fabric.



Since the base of the clip holds the fabric, it is firmly attached.



Sew a button outside of the fabric.



Slight roughness appears on the base of the clip after installation.

MEIRA SENKO MEDICAL INSTRUMENT Mfg. Co., Ltd.

2-11-1, Hamakawado, Kasukabe-shi, Saitama 344-0054 Japan.

Contact : Phone. +81-3-3812-3254 Fax. +81-3-3815-7011 <http://www.mera.co.jp/>