

MERA SUCUUM 009

Thoracic drainage systems






3 colors are available for the unit panel.

You can choose among pink, green or blue according to your application.



TREND A trend function that is useful for monitoring air leakage during drainage of the thoracic cavity



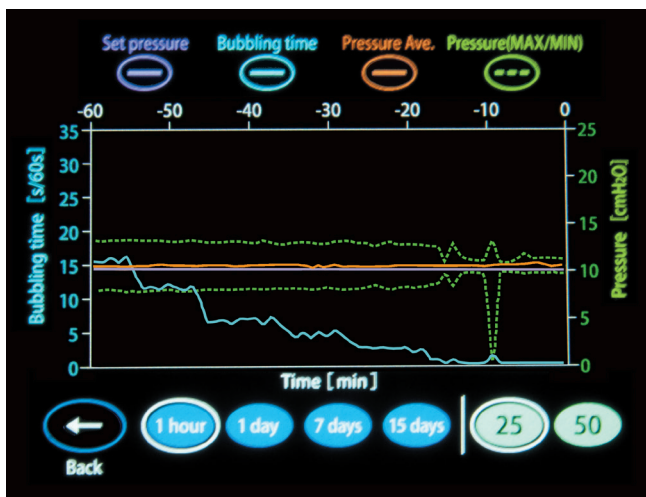


- This unit displays air bubbling time, mean pressure and max/min pressure which are detected with a pressure sensor.
- Such parameters are useful for monitoring the presence of air leakage during patients resting, coughing or taking a deep breath.

USER-FRIENDLY Clear and simple color LCD touch panel

User-friendly design for avoiding operational error

- Periodic time can be set in units from sec. to min. by continuously moving up and down the switch.
- You can customize the screen display to simply indicate desired functions in restricted mode. (Functions include automatic lock, periodic suction, trend and mute/pause for alarm).

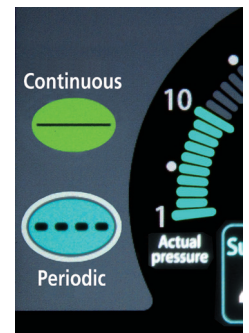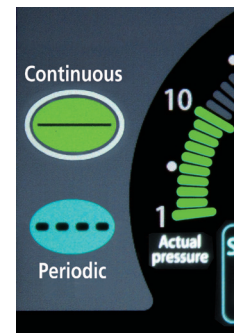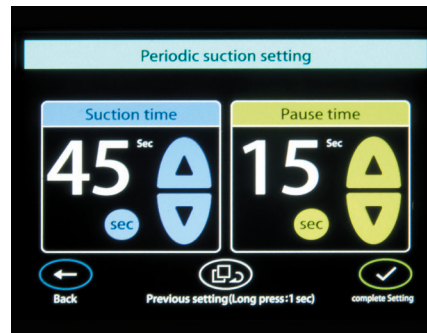

CONTAMINATION / INFECTION CONTROL



Single use drain tank set / 009T-ACC-SDTS

- Suction circuit with PTFE filter (hydrophobic type)

Reusable drain tank set / 009T-ACC-RDTS

- Reusable type of suction circuit with washing, immersive cleaning and disinfection functions

Ⓐ Overflow protection filter/009T-ACC-OPF

Options

Suction circuits



■ 009T-ACC-SDTS



■ 009T-ACC-RDTS

Connecting tubes



■ US-P-1500



■ UL-P-1500

Drain bags



■ AQ-D2-DC



■ AQ-2000-DCSC



■ AQ-CV

Specification

Set pressure Unit of pressure	-1 to -50 hPa · cmH ₂ O · mbar
Maximum suction volume	Not less than 2.0 L/min
Range of periodic suction setting	Suction time: 1 to 59 sec/1 to 60 min Pause time: 1 to 59 sec/1 to 60 min
Rated supply voltage	220 - 240 V AC
Rated supply frequency	50/60 Hz
Battery	12 V DC Nickel-Metal Hydride Battery Continuous operating time: Not less than 60 minutes Charge time: Approximately 4 hours
Power input	60 VA or less
Overall dimensions	250 mm (W) × 200 mm (D) × 480 mm (H)
Weight	4.3 kg

Product List

	Code #	REF	Description	Quantity
Devices	0159800300	MS-009T-ESE(G)	Green	1 set / box
	0159800400	MS-009T-ESE(B)	Blue	
	0159800500	MS-009T-ESE(P)	Pink	
Suction circuits	0159803901	009T-ACC-SDTS	Single use drain tank set	10 pieces/box
	0159803700	009T-ACC-RDTS	Reusable drain tank set	1 piece/box
	0159803801	009T-ACC-OPF	Overflow protection filter	10 pieces/box
Connecting tubes	0159805201	US-P-1500	OD4.8-8.8mm/1500mm	10 pieces/box
	0159806001	UL-P-1500	OD7.4-13.6mm/1500mm	
Drain bags	0159804001	AQ-D2-DC	1000mL	20 pieces/box
	0159804401	AQ-2000-DCSC	2000mL	12 pieces/box
	0159804901	AQ-CV	Check valve	10 pieces/box

Manufacturer



Senko Medical Instrument Mfg. Co., Ltd. Production Planning

2-11-1 Hamakawado, Kasukabe-shi, Saitama 344-0054, Japan
TEL +81-3-3812-3254 / FAX +81-3-3815-7011

●BO-0482-02 ●B-123 ●9/2023 ●0

EC REP European Authorized Representative
MedNet EC-REP GmbH
Borkstrasse 10, 48163 Muenster, Germany
TEL +49 251 32266-0 FAX +49 251 32266-22

